

DISCOVER THE **OXY** EFFECT - NOW ALSO IN **XL**

## Incidin™ OxyFoam S & OxyWipe S

Sporicidal ready-to-use cleaner and disinfectant with Hi-speed H<sub>2</sub>O<sub>2</sub>™

The advanced formulation of Incidin™ OxyFoam S & Incidin™ OxyWipe S provides the benefits of a patented\* hydrogen peroxide solution with broad spectrum efficacy.

They are effective against a wide range of pathogens and leave no active residues due to their rapid evaporation rate.

Incidin™ OxyFoam S & OxyWipe S provide:

- ▲ Hi-speed H<sub>2</sub>O<sub>2</sub>™
- ▲ Active ingredient decomposing into water and oxygen
- ▲ Sporicidal & virucidal offering for all surfaces
- ▲ Excellent material compatibility



# Incidin™ OxyFoam S & OxyWipe S

Sporicidal ready-to-use cleaner and disinfectant with Hi-speed H<sub>2</sub>O<sub>2</sub>™

**3 MIN<sup>1</sup>** Sporicidal against **C.difficile** (SUSPENSIONS TEST) **2 MIN<sup>1</sup>** Virucidal (CLEAN CONDITIONS) **1 MIN<sup>1</sup>** Bactericidal (EN 13727 / 16615)




| Features          | Benefits   |
|-------------------|--|
| <b>EFFECTIVE</b>  | Broad spectrum efficacy<br>Short contact times                                       |
| <b>SAFE</b>       | Active ingredient decomposes into water and oxygen. Excellent material compatibility |
| <b>CONVENIENT</b> | Cleans & disinfects in one step<br>Ready to use formats                              |

## MICROBIOLOGICAL AND VIROLOGICAL TEST RESULTS

| EFFICACY UNDILUTED  | OxyFoam S CONTACT TIME                                | OxyWipe S CONTACT TIME |
|---|---|------------------------|
| Disinfection of surfaces acc. to VAH method 2015 (incl. EN 16615), dirty conditions |   |                        |
| Bactericidal, yeasticidal   | 5 min   | 15 min                 |
| Fungicidal  | 5 min   | 30 min                 |
| Mycobactericidal  | 15 min  | 30 min                 |
| Tested acc. RKI recommendation 01/2004*   |   |                        |
| Effective against enveloped viruses acc. to RKI recommendation 01/2004*             |   |                        |
| Noro (MNV)*   | 30 min  | 60 min                 |
| Adeno*  | 30 sec  | 1 min                  |
| Polyoma SV40*   | 30 sec  | 30 sec                 |
| Euro-Norms (clean & dirty conditions)   |   |                        |
| EN 13727 (bactericidal)   | 1 min   | 2 min                  |
| EN 13624 (yeasticidal)  | 1 min   | 15 min                 |
| EN 13624 (fungicidal)   | 5 min   | 30 min                 |
| EN 14348 (tuberculocidal, mycobactericidal)   | 5 min   | 30 min                 |
| EN 14476 (virucidal)  | 2 min** / 30 min                                      | 30 min**               |
| EN 14476 Adeno  | 30 sec  | 30 sec                 |
| EN 14476 (limited spectrum virucidal)   | 1 min (clean conditions)<br>15 min (dirty conditions) | 15 min                 |
| EN 14476 Noro   | 1 min (clean conditions)<br>15 min (dirty conditions) | 15 min                 |
| EN 13704 (sporicidal)**   | 30 min  | 60 min                 |
| EN 13704 (C. difficile)**   | 3 min   | 15 min                 |
| Euro-Norms (dirty conditions) - 4-field-test  |   |                        |
| EN 16615 (bactericidal)   | 1 min   | 2 min                  |
| EN 16615 (yeasticidal)  | 1 min   | 1 min                  |
| EN 16615 (fungicidal, tuberculocidal, mycobactericidal)                             | 5 min   | 5 min                  |
| Sporicidal (method EN 16615)**  | 15 min  | 30 min                 |
| C. difficile (method EN 16615)**  | 5 min   | 5 min                  |




\* DVV 2015 (suspension test) \*\* clean conditions

## INCIDIN™ OXYFOAM S INSTRUCTIONS FOR USE

- 1 Wet surface
- 2 Wipe the surface ensuring it is completely wet
- 3 Let dry

---

- 1 Apply to clean wipe
- 2 Wipe the surface ensuring it is completely wet
- 3 Let dry

**EUROPEAN HEADQUARTERS:**  
**ECOLAB EUROPE GMBH**  
 Richtstrasse 7  
 8304 Wallisellen  
 Switzerland  
 +41(0) 44-877-2001  
 www.ecolab.com



**ECOLAB DEUTSCHLAND GMBH**  
 Ecolab-Allee 1  
 40789 Monheim am Rhein  
 +49 (0) 2173-599-1900



EFFECTIVE.  
 SAFE.  
 CONVENIENT.

### APPLICATION AREA

Fast acting, sporicidal ready-to-use surface cleaning and disinfection



▲ acc. Directive 93/42/EEC: for medical devices.



▲ Biocide: for general surfaces.

### INSTRUCTIONS FOR USE

**Incidin™ OxyFoam S:** Wet surface, ensuring the surface is completely wet. Wipe the surface with a clean wipe and leave to dry. Alternatively, apply the product to a clean wipe, wipe the surface and leave to dry. Do not rinse.

**Incidin™ OxyWipe S:** Take single wipe. Wet surface thoroughly, ensuring contact with the entire surface. Let it dry and do not rinse. Close lid after use.

The use of a cleaning and disinfection product does not replace regular cleaning. For reprocessing of medical devices always observe manufacturers' instructions for use, incl. material compatibility. Terminal disinfection of semi-critical medical devices (i.e. probes & TEE probes): If immersion is not possible, disinfect the device by wiping as described above. Rinse thoroughly with water of drinking quality after use in this case. Do not use on surfaces sensitive to oxidative agents such as marble, copper and brass.

### INGREDIENTS

100 g contains the following active:  
**Incidin™ OxyFoam S:** 1.5 g Hydrogen peroxide  
**Incidin™ OxyWipe S:** 1.5 g Hydrogen peroxide

### PACKAGING AND ORDERING INFORMATION

| PRODUCT                                       | OUTER SIZE           | ORDER CODE |
|---|----------------------|------------|
| Incidin™ OxyFoam S                            | 6 x 750ml            |            |
| Incidin™ OxyFoam S                            | 2 x 5L               |            |
| Incidin™ OxyWipe S                            | 6 x 100 (20 x 20 cm) |            |
| Incidin™ OxyWipe S XL                         | 6 x 50 (25 x 37 cm)  |            |
| drain cock (DIN 51) for 5L canister           | 1 pcs                | 10033873   |
| wall mounted dispenser for Incidin™ OxyWipe S | 6 x 1 pcs            | 10033877   |

### STORAGE

Product should be stored in the original container below 25 °C in a cool, dry location out of direct sunlight.

Shelf life after first opening:  
**Incidin™ OxyFoam S:** End of expiry date  
**Incidin™ OxyWipe S:** 3 months

References  
 1. Incidin™ OxyFoam S

USE BIOCIDES SAFELY. ALWAYS READ THE LABEL AND PRODUCT INFORMATION BEFORE USE. FOR PROFESSIONAL USE ONLY.

